

# Veritas<sup>®</sup> Collagen Matrix

for  
**Breast Reconstruction**



Works in  
Conjunction  
with the Body's  
Natural Healing  
Process


Synovis<sup>®</sup>  
Surgical Innovations

## *Provides an Ideal Scaffold for Soft Tissue Repair*

Veritas® is a strong, acellular, biocompatible, remodelable implant that provides an ideal scaffold for soft tissue repairs.

### **Veritas is...**

1. Accepted by the body without a foreign body response
2. Infiltrated by cells and blood vessels
3. Part of the natural healing process; allowing the body's natural remodeling to occur and over time becomes indistinguishable from the adjacent host tissue.

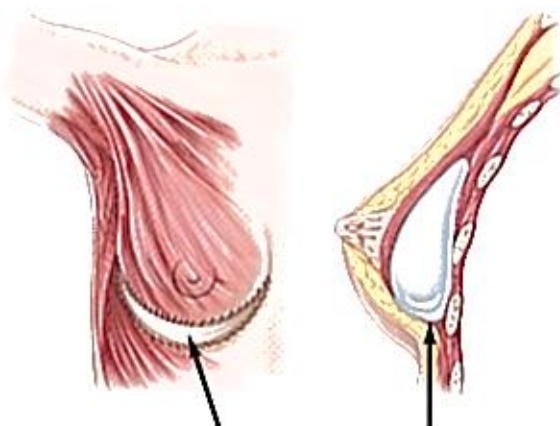

**Veritas® Collagen Matrix**


Veritas may be used for tissue expander and implant support

### **Benefits of Veritas for Breast Reconstruction:**


- ✓ Provides excellent support for tissue expansion
- ✓ Strong and Pliable
- ✓ Helps keep implant secure in the pocket
- ✓ Remodels for rapid integration into surrounding tissues
- ✓ Easy to use
  - Ready right out of the package
  - No rehydration required
  - Easy to cut and sew
  - Stored at room temperature
  - 3 year shelf life



# Veritas® for Tissue Expansion

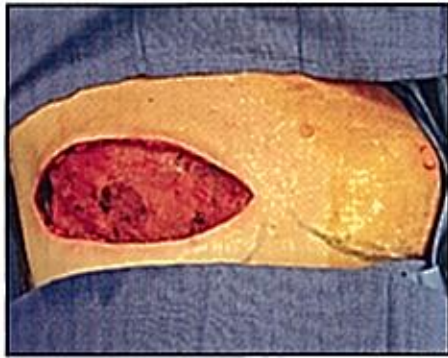


Step 1: Inframammary fold (IMF) drawn before mastectomy

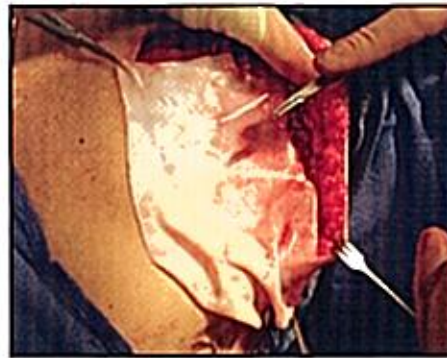


Step 2: Lift your inferior flap down to IMF




Step 3: Veritas® sewn to inferior IMF after measuring size

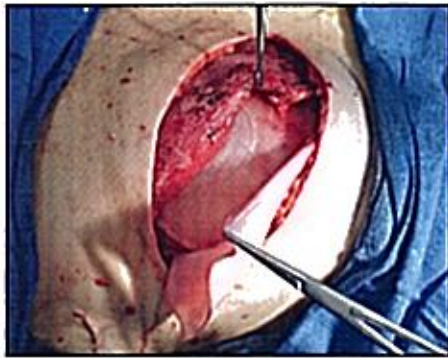

Step 4: Place appropriate implant submuscular under sling and pectoralis major muscle

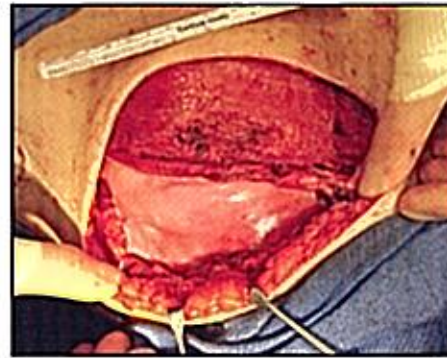

Step 5: Veritas sewn to pectoralis major muscle

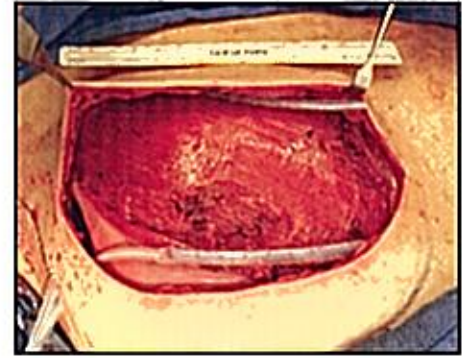

Step 6: Two J-P 7mm drains



Step 7: Skin closed in 2 layers



First expansion postoperative day 14



Almost maximally expanded



Oblique view: Ready for permanent saline or gel implant



Side view: Ready for permanent saline or gel implant



Tissue fully expanded and ready for implant at two months

# Convenient Sizes Available

12 cm x 25 cm

10 cm x 16 cm

8 cm x 14 cm

6 cm x 8 cm

4 cm x 15 cm

4 cm x 7 cm

2 cm  
x 8 cm

## Ordering Information

RM-0208	2 cm x 8 cm
RM-0407	4 cm x 7 cm
RM-0415	4 cm x 15 cm
RM-0608	6 cm x 8 cm
RM-0814	8 cm x 14 cm
RM-1016	10 cm x 16 cm
RM-1225	12 cm x 25 cm

To order or for more information,  
contact Synovis Surgical Innovations at  
800.487.9627  
or your local sales representative.

**With Veritas<sup>®</sup> you feel  
Confidence in your  
Results...based on  
Exceptional  
Performance**

- 10,000+ implants in soft tissue repair
- Over 25 years of biologic tissue expertise
- More than 150,000 implants with Veritas<sup>®</sup> Collagen Matrix technology

**Synovis<sup>®</sup>**

Surgical Innovations

A Division of Synovis Life Technologies, Inc.

2575 University Ave., W.  
St. Paul, MN 55114-1024 USA  
651.796.7300 800.255.4015  
651.642.9018 (fax)

