

Endo-Sock™

Laparoscopic Retrieval Pouch

Directions for Use



REF 81044

Intended Use

The Endo-Sock is designed to assist in the retrieval of infectious or suspicious tissue and gall stones during laparoscopic procedures.

Sterile STERILE EO

Contents are sterile if package is unopened and undamaged.

Disposable (2)

Single-use only. Store in a cool, dry place.

Cautions

U.S. Federal law restricts this device to sale by or on the order of a physician. Read all instructions prior to use.

Warnings

Laparoscopic procedures should only be performed by persons with adequate training in laparoscopic techniques. Consult the medical literature relative to techniques, complications and hazards prior to performing any laparoscopic procedure.

- Use of the Endo-Sock™ device requires a 10mm or larger trocar sleeve.
- Endo-Sock™ is packed in a sheath and should not be deployed prior to insertion. Premature deployment of the pouch from the sheath may damage the pouch or render the pouch inoperative. The pouch cannot be repacked.
- Endo-Sock™ must be inserted through the trocar sleeve as packaged.
- Excess traction may cause rupture of the Endo-Sock™ bag allowing the contents to spill and contaminate the wound and/or peritoneal cavity.

Directions for Use

- 1 Following recommended technique, prepare the patient for laparoscopic surgical procedure.
- 2 Using sterile technique, remove the retrieval pouch from its sterile package.
- 3 **To Insert Endo-Sock™**—Grasp the silver shaft just before the stopper. Insert the Endo-Sock™ sheath through a 10mm or larger trocar sleeve. Push shaft forward until the stopper makes contact with the trocar head.
- 4 **To Deploy Pouch**—Under direct visualization, push the handle and spool forward toward the stopper to deploy and open the pouch.
NOTE: The handle, spool and silver shaft move forward together, in one motion.
- 5 Grasp tissue to be removed and put into pouch.
- 6 **To Close Pouch**—With one hand grasping the spool, grasp the handle with the other hand and pull it away from the spool. The pouch will close.
- 7 **To Remove Pouch with Tissue**—
 - A Pull closed pouch towards the distal end of the trocar sleeve by pulling the handle toward the user.
 - B Pull the trocar sleeve containing the sheath out of the body until the pouch begins to exit from the port opening.
 - C Grasp the pouch and determine the best method to remove pouch with tissue. This may include removal of stones, morcellation of tissue, opening of port incision, etc.
- 8 **Product Disposal**—Dispose of Endo-Sock™ according to local hazardous waste practices, once tissue and/or gallstones have been removed.

Distributed by:

CooperSurgical

Trumbull, CT 06611 USA

Phone: (800) 243-2974

Fax: (800) 262-0105